


# EPIDINO

Deutsch

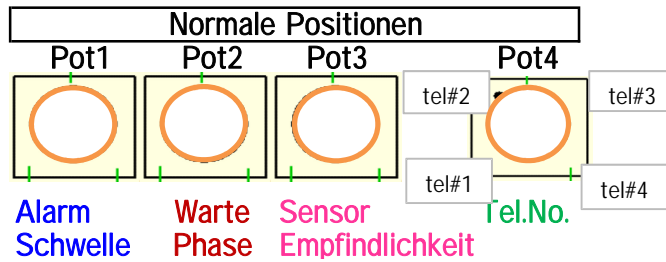

[www.dk2os.net](http://www.dk2os.net)

Nach wieviel Sekunden Bewegungs-Registrierung der Alarm ausgelöst werden soll.


Wie lange auf die nächste Bewegung gewartet wird. Kommt keine Bewegung mehr, so schaltet der Timer aus.

Einstellung der Sensorempfindlichkeit

DIP switch - Wenn Pot4 in Pos. tel#2		
Nr. 1	Nr. 4	
OFF	ON	tel#2 immer, tel#1 je nach Timerzeit
ON	ON	tel#2 immer, tel#1 auch
OFF	OFF	tel#2 immer, tel#1 nicht

Zeitschaltuhr muss auf "AUTO" gestellt sein.

English

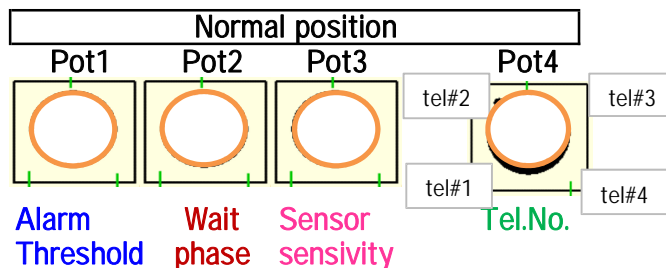

[www.dk2os.net](http://www.dk2os.net)

After how many seconds of motion registration the alarm should be triggered.

How long to wait for the next movement. If there is no movement, the timer switches off.

Adjustment of sensor sensitivity

DIP switch - When Pot4 is in Pos. tel#2		
No. 1	No. 4	
OFF	ON	tel#2 always, tel#1 in accordance with the timer
ON	ON	tel#2 always, tel#1 also
OFF	OFF	tel#2 always, tel#1 not

Timer must be in AUTO position

Potis anfangs in diese Positionen stellen und dann Pot1, 2 und 3 erforderlichenfalls veraendern.

Set pots into this position first, and then adjust Pot1, 2 and 3 if necessary.

